


ONYX® Deskset HD Remote Control Reference Card

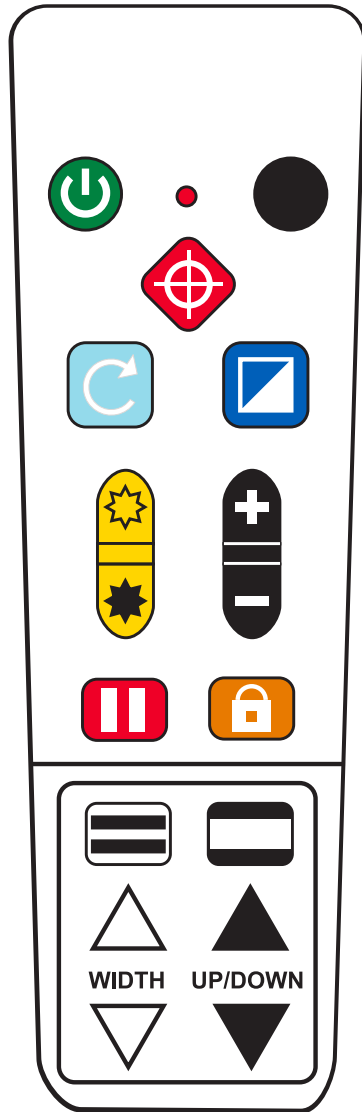

Power (Green)
Powers unit on and off



Freeze Frame (Red)
Freezes the current image displayed on the screen



Advanced (Black)
Use in combination with other buttons to perform advanced tasks. Always press and hold this button first, and then press another button to perform the task.



Focus Lock (Orange)
Turns Auto Focus on and off



Find (Red)
Displays crosshairs and zooms out to help aim the camera



Reading Lines (White and Black)
Displays two horizontal or vertical reading lines



Flip (Light Blue)
Cycles through four different picture orientations



Mask (Black and White)
Displays shades or masks to cover parts of the screen



Color Mode (Blue)
Cycles through different color modes



Width (White)
Changes spacing between reading lines, shades, or masks



Brightness (Yellow)
Changes screen brightness



Up and Down (Black)
Moves reading lines, shades, or masks on the screen



Zoom (Black)
Zooms camera view in and out



Advanced + Power
Hold for approximately six seconds to enter or exit Configuration mode. While in this mode, you can add, remove, or reset color modes.