


<p>(1) BRIGHTNESS (Yellow dial)</p>	<p>Turn clockwise to increase screen brightness.</p> <p>Turn counterclockwise to decrease screen brightness.</p>
<p>(2) FREEZE FRAME (Red button)</p>	<p>Press to freeze the image that is displayed on the screen.</p> <p>Press again to return to live view.</p> <p>Press and hold for approximately six seconds to enter or exit Review mode. Review mode lets you view or delete images saved on an SD card.</p>
<p>(3) MAGNIFICATION (Black dial)</p>	<p>Turn to zoom in and increase magnification or zoom out and decrease magnification</p>
<p>(4) FIND (Orange button)</p>	<p>Press and hold to zoom out and display crosshairs on the screen to help you reposition the object under the camera.</p> <p>Release to return to the previous magnification level.</p>
<p>(5) COLOR MODE (Blue dial)</p>	<p>Turn to cycle through color modes.</p>
<p>(A) ADVANCED (Black button)</p>	<p>Use in combination with other controls to access and perform advanced features.</p> <p>See Advanced Functions for more information.</p>

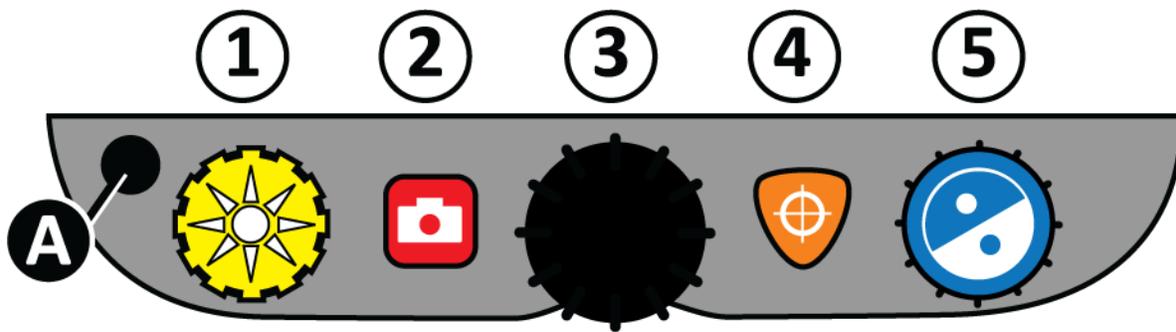

The black **ADVANCED** button (**A**) is used with other control panel buttons and dials to access and perform more features and operations.

Always **press and hold** the **ADVANCED** button (**A**) first, and then press or turn the other buttons or dials to perform the advanced task.

For example, to dim or brighten the LED reading light, first **press and hold** the **ADVANCED** button (**A**), and then turn the yellow **BRIGHTNESS** dial (1).

<p>(A + 1) LED Brightness (Yellow dial)</p>	<p>Turn to adjust the LED reading light brightness.</p>
<p>(A + 2) Save Image (Red button)</p>	<p>Press and hold to save the current image to an SD Card.</p> <p>Release when the Save icon appears.</p> <p>Images can only be saved when an SD Card is in the SD Card slot.</p>
<p>(A + 3) Lines and Masks (Black dial)</p>	<p>Turn to cycle through and display horizontal lines, horizontal masks, vertical lines, vertical masks, and no lines or masks on the screen.</p>
<p>(A + 4) Focus Lock (Orange button)</p>	<p>Press and hold to turn Focus Lock on and off.</p> <p>Release when the Lock or Unlock icon appears.</p>
<p>(A + 5) Move Lines or Masks (Blue dial)</p>	<p>Turn to adjust the spacing between lines or masks.</p>
<p>(A) Configuration Menu (Black button)</p>	<p>Press and hold for approximately 10 seconds to enter or exit Configuration mode.</p>