


TOPAZ PHD Focus Lock Update


Focus lock keeps the camera focused on the item you are viewing instead of trying to automatically focus on your hand, or any other item that passes in front of the camera. This is useful when working or writing directly under the camera.

To use Focus Lock, do the following:

1. Press and **hold** both the black **ADVANCED** button (on the left side of the control panel) and the orange **FIND** button .

Release the buttons when the Lock icon  appears on the screen.

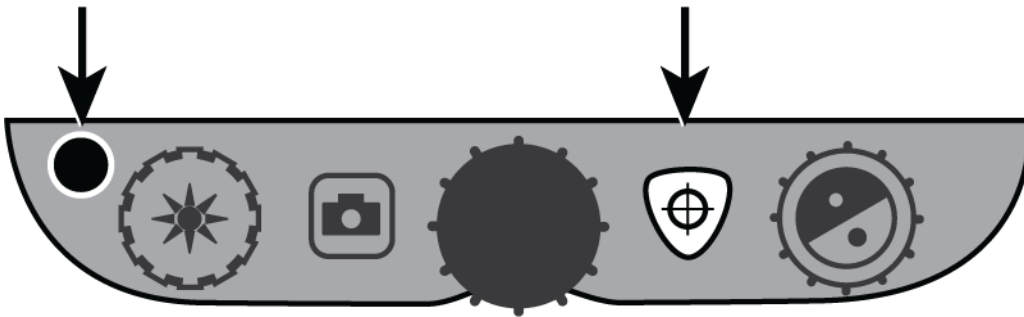

2. Repeat step 1 to turn off Focus Lock  and restore automatic focus.

New Button Assignment for Configuration Mode

With the addition of Focus Lock, the button sequence used to enter and exit Configuration mode has changed. Instead of using both the **ADVANCED** and **FIND** buttons, you now use only the **ADVANCED** button.

To enter and exit Configuration mode, do the following:

- Press and **hold** the black **ADVANCED** button (on the left side of the control panel) for approximately 10 seconds

